

# EZSIMULATOR INSTALLATION QUICK GUIDE




## APP INSTALLATION

The eZSimulator app is available through the Microsoft Windows Store.

To install the app, open the Store and search for „eZSimulator“ or use the QR code.

In the Store window click „Get“ to obtain the app and, if necessary, „Install“ to actually install the app.

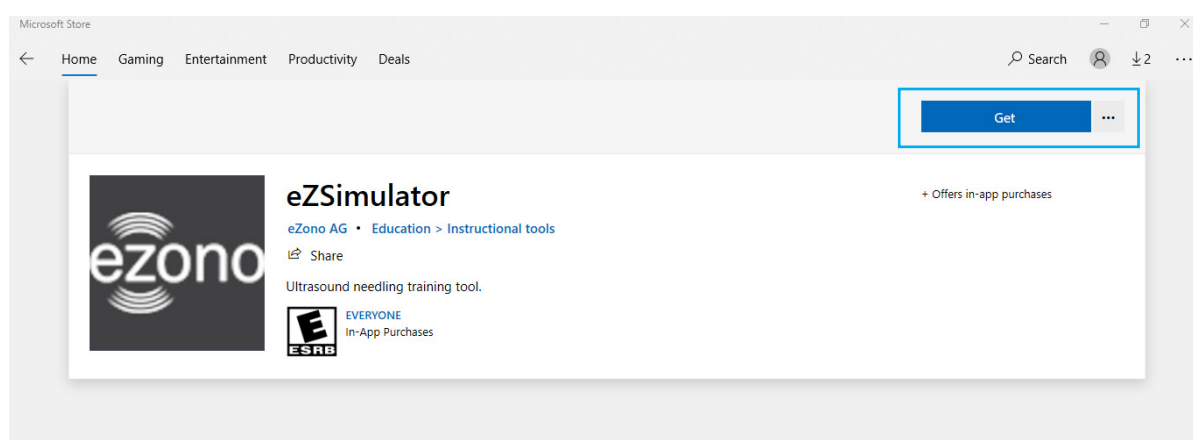

## PROBE INSTALLATION

The eZGuide USB Probe requires a driver to function. The driver is provided through Windows Update.

To install the driver, just connect the probe to an unused USB port and wait until Windows has installed the driver.

Windows will show a notification when the driver has been installed.

The successful installation of the driver can be checked in the ‚Bluetooth & other devices‘ section of Windows Settings/ Devices.

