



EZGUIDE® USB PROBE

USER MANUAL

TABLE OF CONTENTS

1.	WELCOME.....	3
2.	INSTALLATION.....	4
3.	SUPPORT.....	5
4.	SAFETY AND LIABILITY.....	5
5.	DISPOSAL.....	5
6.	COMPLIANCE.....	6

1. WELCOME

1.1. OVERVIEW EZGUIDE USB PROBE

The eZGuide® USB Probe is a training probe. In combination with a compatible training software various ultrasound-supported and ultrasound-navigated procedures can be trained. While the probe features an innovative needle navigation system, it does not contain an ultrasound array and therefore cannot emit or receive ultrasound.

- ✔ Windows 10 supported.
- ✔ No ultrasound system necessary.
- ✔ Needle navigation support.
- ✔ Time-independent training.
- ✔ Cost-effective.

NOTE: The eZGuide® USB Probe is only a training device. It is not intended for human use.


2. INSTALLATION

The eZGuide® USB Probe can be used on any Windows 10 PC/Laptop. To correctly function the probe requires a driver to be installed.

This driver is installed automatically through Windows Update. When connecting the probe first time to a PC, Windows will search for the driver and install it.

Please wait until the driver got installed, Windows will notify you once the probe is ready to be used.

Updates to the driver will also be published though Windows Update and applied automatically though the update process.

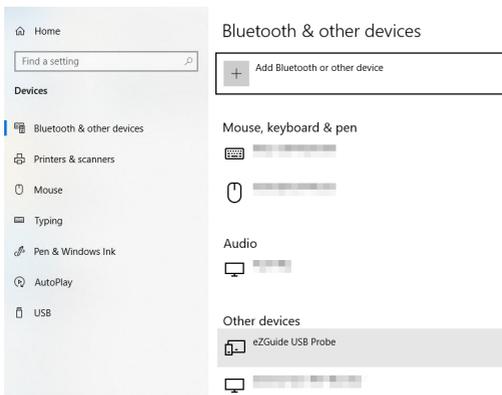

Picture 01: Windows settings

If the driver for the eZGuide® USB Probe was installed correctly and the probe is connected to the computer, it will be listed under *Bluetooth & other devices* section in the system settings.

3. SUPPORT

If you have any questions, do not hesitate to contact eZono.
eZono AG - Spitzweidenweg 32
07743 Jena, Germany
Phone: +49 (0) 3641 876 1740
Fax: +49 (0) 3641 876 1759
Email: education@ezono.com

4. SAFETY AND LIABILITY

DISCLAIMER:

eZSimulator components should only be used for training purposes. Any manipulation of eZSimulator components will lead to the exclusion of liability on the part of eZono.

5. DISPOSAL



This symbol on the product is in accordance with the European Union's Waste Electrical and Electronic Equipment Directive (WEEE). This symbol indicates that this product must be recycled/disposed separately from other household waste. It is the end user's responsibility to dispose this product by taking it to a designated WEEE collection facility for the proper collection and recycling of the waste equipment.

6. COMPLIANCE

6.1. FCC COMPLIANCE STATEMENT



This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/ TV technician for help.

WARNING: Changes or modifications to the product not expressly approved by eZono or FCU could void your right to use or operate your product.

US Agent:
1950 Lefthand Creek Lane Longmont, CO 80501
Phone: +1 303 834 8413

6.2. EUROPEAN COMPLIANCE STATEMENT



A copy of the EU Declaration of Conformity is available at:
<https://www.ezono.com/en/documents/>

6.3. KOREAN COMPLIANCE STATEMENT



R-R-FCu-eZSim-P1L

FCU Co., Ltd.
B-620/621, Biz Center, 17, Techno 4-ro
Yuseong-gu, Daejeon 34013, Republic of Korea